



MODEL#S34481

Rev: 0

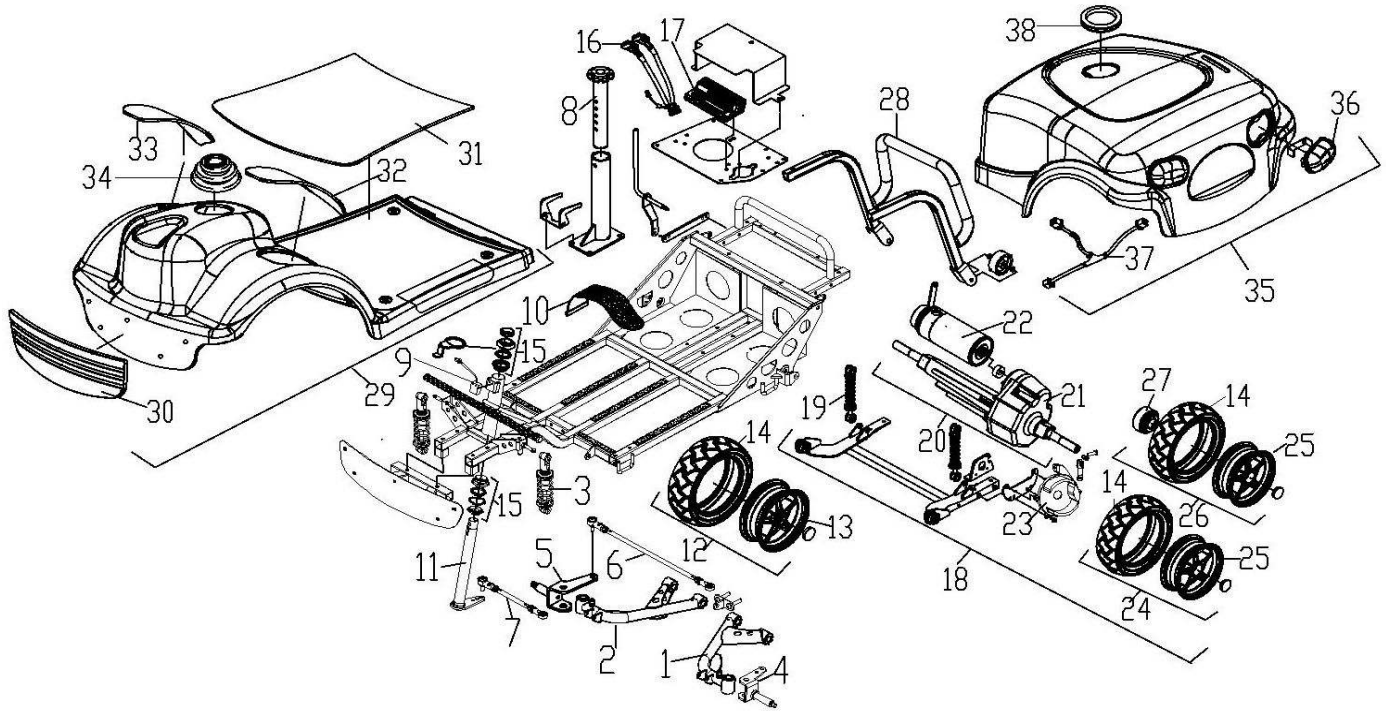

REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION
1	30300424	FRONT SUSPENSION ASS'Y,L	16	63370420	POWER HARNESS,DS160/S347	29	30516112	FRONT SHROUD ASSY., OCEAN BLUE, S344
2	30300425	FRONT SUSPENSION ASS'Y,R	17	64510008	CONTROLLER, DYNAMIC RHINO DS160 RHINO2	29	30510795	FRONT SHROUD ASSY., EXTEND, CANDY APPLE RED, S344
3	41700113	SUSPENSION,165L,BLACK (150LBS/IN)	18	30400293	REAR FRAME ASS'Y	30	34200100	FRONT BUMPER (PU FOAM)
4	30303101	FRONT WHEEL AXLE ONLY, LH	19	41700105	SUSPENSION BLACK/165L(550LBS/IN)	31	43701010	CARPET, GRAY, S347
5	30303102	FRONT WHEEL AXLE ONLY, RH	20	62001308	TRANSAXLE ASSY., M802AH-7	32	43701008	LEFT-CARPET, GRAY, S347
6	30302009	ADJUST SHAFT, LONG, ASS'Y	21	62203003	TRANSAXLE,M802	33	43701009	RIGHT-CARPET,GRAY, S347
7	30302010	ADJUST SHAFT, SHORT 10*175L	22	62101302	MOTOR,M8AH-7	34	44600110	RUBBER BOOT,SEAT POST
8	32100025	UPPER SEAT TUBE ASS'Y.	23	41700214	BRAKE (\$ 100)	35	30519287	REAR SHROUD ASSY.(WIDE), SILVER WHITE, S344
9	63742002	SENSOR, HALF SPEED	24	32600001	100/65 9" REAR WHEEL ASSY BLACK, BLACK	35	30520074	REAR SHROUD ASSY.(WIDE), CHAMPAGNE, S344
10	43700203	BATTERY STRAP, 50*660L	25	42500501	RIM, ALU., 9" REAR WHEEL	35	30516103	REAR SHROUD ASSY.(WIDE), OCEAN BLUE, S344
11	30303236	TURN TUBE COMPLETE	26	32600006	9" REAR WHEEL ASSY BLACK LEFT WITH BRAKE	35	30510442	REAR SHROUD ASSY.(WIDE), CANDY APPLE RED, S344
12	33100688	100/65-9" FRONT WHEEL ASSEMBLY , BLACK	27	41200439	BRAKE DRUM (φ100)	36	61200144	REAR LAMP, COMBINATION
13	42500500	RIM, ALU., 9" FRONT WHEEL	28	34203129	REAR BUMPER (ED) S344	37	63444302	WIRE LINK BRAKE & INDICATOR LIGHTS,220L,S137
14	44600402	TYRE 100/65-9, BLACK, S344 WIDE VERSION	29	30519306	FRONT SHROUD ASSY., EXTEND, SILVER WHITE, S344	38	44600103	RUBBER BOOT, SEAT POST
15	53400023	BEARING ONLY, FORK 7 PCS/SET	29	30520081	FRONT SHROUD ASS'Y, CHAMPAGNE GOLD			

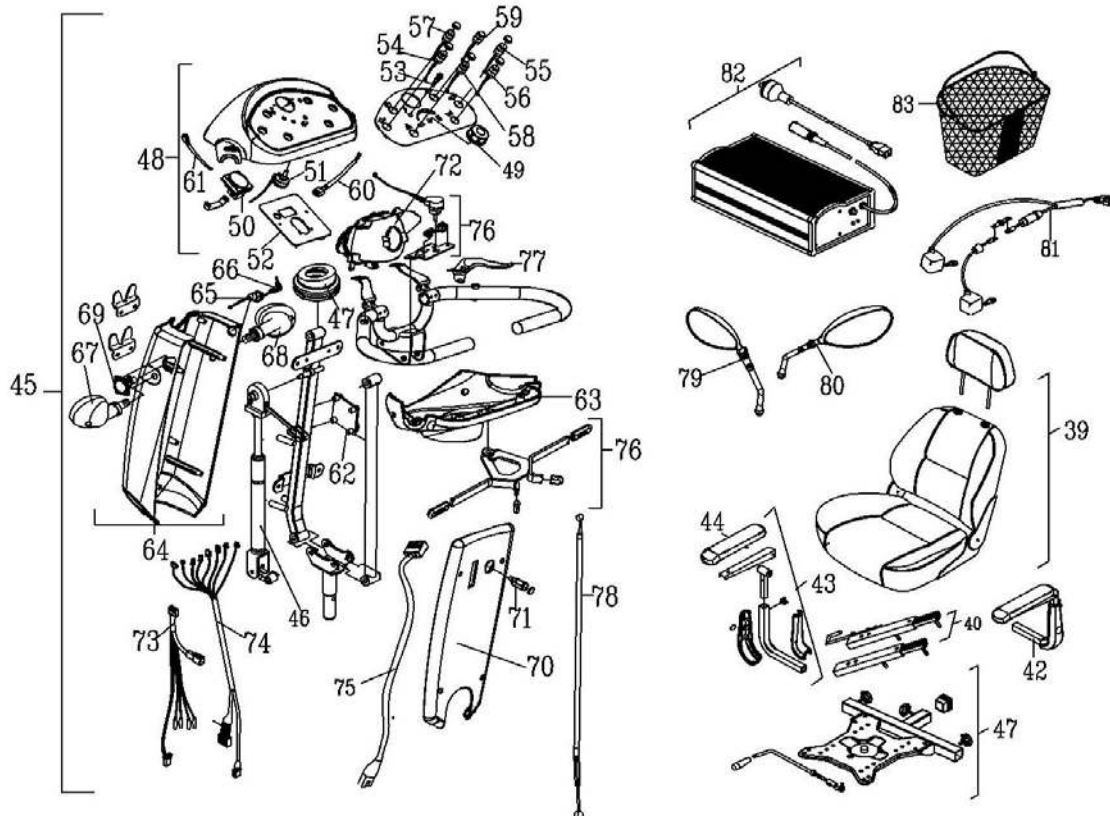

REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION
39	32222085	CAPTAIN SEAT ONLY,20.5W*18D*20H,W/RECLINE	53	61200070	POWER INDICATOR, L-610	67	61200050	INDICATOR UNIT, LEFT (24V/5W)
40	41700077	SLIDE MECH. ASS'Y., WITH RELEASE LEVER	54	33700220	LEFT DICTATOR LIGHT SWITCH SET,150L,P9L	68	61200051	INDICATOR UNIT, RIGHT (24V/5W)
41	32100039	SEAT BASE WITH RELEASE LEVER ASS'Y	55	33700221	RIGHT DICTATOR LIGHT SWITCH SET,150L,P9R	69	33710008	CHARGER SOCKET SET
42	31000089	ARMREST ASS'Y., LH	56	33700223	HORN SET,100L,P6	70	30519492	REAR DASHBOARD ONLY, SILVER WHITE, 12V
43	31000090	ARMREST ASS'Y., RH	57	33700224	HEADLIGHT SWITCH,150L,P8	70	30520094	REAR DASHBOARD ONLY, CHAMPAGNE GOLD, 12V
44	43700401	ARM PAD,BLACK	58	33700225	HAZARD LIGHT SET,100L,P14	70	30516155	REAR DASHBOARD ONLY, OCEAN BLUE, 12V
45	33519152	TILLER ASS'Y., PARALLEL, SILVER WHITE	59	33700255	HALF SPEED SET,S139	70	30510414	REAR DASHBOARD ONLY, CANDY APPLE RED, 12V
45	33520028	TILLER ASS'Y., PARALLEL, CHAMPAGNE GOLD	60	63545203	CONNECTING WIRE, SPEED POT,180L	71	61138003	CABLE, PLUG.
45	33516032	TILLER ASS'Y., PARALLEL, OCEAN BLUE	61	63947101	WIRE,FUEL GAGE,S139	72	61200148	HEAD LIGHT ASS'Y,NE1-24V/18W
45	33500146	TILLER ASS'Y., PARALLEL, CANDY APPLE RED	62	61100069	POWER CONVERT BOARD	73	63445209	WIRE,HEAD LIGHT W/ THRN LIGHT,250L
46	41300570	ELEVATING CYLINDER	63	30519266	LOWER DASHBOARD ONLY, SILVER WHITE	74	63180063	MAIN CABLE WIRE FOR TILLER, S3478 SW
47	44689071	RUBBER BOOT, NBR, φ112*61.4, SW	63	30520061	LOWER DASHBOARD ONLY, CHAMPAGNE GOLD	75	63742003	SENSOR, HALF SPEED
48	33519151	UPPER DASHBOARD ASS'Y., SILVER WHITE	63	30516085	LOWER DASHBOARD ONLY,OCEAN BLUE	76	33500110	THROTTLE LEVER ASS'Y.
48	33520033	UPPER DASHBOARD ASS'Y., CHAMPAGNE GOLD	63	30510903	LOWER DASHBOARD ONLY, CANDY APPLE RED	77	42700041	LEVER BRAKE, RH
48	33516031	UPPER DASHBOARD ASS'Y., OCEAN BLUE	64	30519267	FRONT DASHBOARD ASS'Y., SILVER WHITE	78	41700210	BRAKE WIRE SET.
48	33510897	UPPER DASHBOARD ASS'Y., APPLE RED	64	30520062	FRONT DASHBOARD ASS'Y., CHAMPAGNE GOLD	79	46700006	BACK MIRROR, LH
49	61180107	KNOB,RN-99C	64	30516087	FRONT DASHBOARD ASS'Y., OCEAN BLUE	80	46700008	BACK MIRROR, RH
50	61220012	INDICATOR, BATTERY LEVEL	64	30510904	FRONT DASHBOARD ASS'Y., CANDY APPLE RED	81	63270402	BATTERY CONNECTOR WIRING
51	61020006	POTENTIOMETER., VR30K, COSMOS RV24	65	63700036	KEY SWITCH W/CONNECTING WIRE, S830	82	33700462	CHARGER,HP0240WB 24V 8A,WITH CORD
52	61108225	PCB MODULAR, DASHED BOARD, PIR113, WIRE LINK BRAKE	66	33700217	KEY SET	83	30300296	BASKET, PLASTIC