



MODEL#S941--SW

Rev: 0

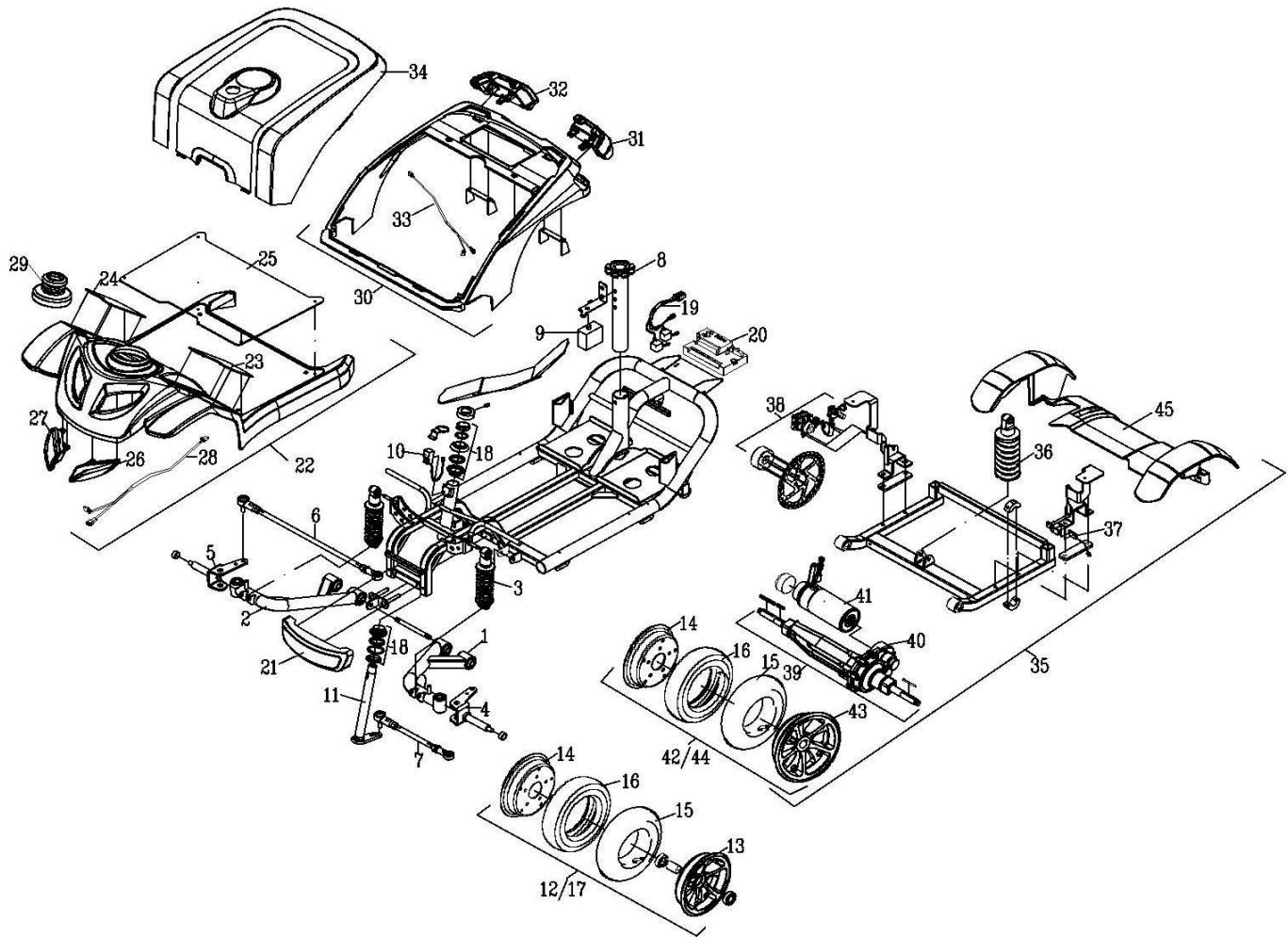

REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION
1	30300403	FRONT SUSPENSION ASSY(L)	17	33100776	FRONT WHEEL ASSY(R)	31	61200197	REAR LIGHT ASSY(L)
2	30300404	FRONT SUSPENSION ASSY(R)	18	53400023	BEARING ONLY, FORK 7 PCS/SET	32	61200198	REAR LIGHT ASSY(R)
3	41700290	SUSPENSION(HYDRAULIC),165L,150LBS/IN, YELLOW	19	63370439	POWER HARNESS FOR S941	33	63444302	WIRE LINK BRAKE & INDICATOR LIGHTS,220L,S137
4	30303244	FRONT WHEEL AXLE ASSY, LH	20	64510007	CONTROLLER, DYNAMIC DS120 RHINO2	34	30500280	BATTERY SHROUD ASSY
5	30303245	FRONT WHEEL AXLE ASSY, RH	21	43690195	FRONT BUMPER	35	30400346	REAR FRAME ASSY
6	30302009	ADJUST SHAFT, LONG, ASSY	22	30541064	FRONT SHROUD ASSY, BRIGHT RED, S941, ABS	36	41780025	SUSPENSION(HYDRAULIC), 180L, 150LBS/IN, RED
7	30302013	ADJUST SHAFT, SHORT φ10*175L(ZP)	22	30519444	FRONT SHROUD ASSY, SILVER	37	61090063	SPRING SWITCH,PS8-23
8	32100025	UPPER SEAT TUBE ASSY.	23	44689055	RUBBER MAT, BLACK, FRONT(L)	38	33200153	REAR DISK BRAKE ASSY
9	61230135	BREAKER 80A	24	44689056	RUBBER MAT, BLACK, FRONT(R)	39	62009064	TRANSAXLE ASSY
10	63748101	HARNESS, HALF SPEED SWITCH	25	44689057	RUBBER MAT, BLACK	40	62203037	TRANSAXLE
11	30303258	TURN TUBE COMPLETE (ED)	26	61200195	HEAD LIGHT ASSY(L)	41	62109043	MOTOR, M4QW-21
12	33100775	FRONT WHEEL ASSY(L)	27	61200196	HEAD LIGHT ASSY(R)	42	32600535	REAR WHEEL ASSY(L)
13	41280218	RIM, 6" WHEEL	28	63453015	HEAD LIGHT CABLE	43	41280219	RIM, 6" WHEEL
14	41280212	STEEL CENTER	29	44600317	RUBBER BOOT, NBR, φ114*65, EU	44	32600536	REAR WHEEL ASSY(R)
15	44689073	INNER TUBE ONLY	30	30541065	REAR SHROUD ASSY, BRIGHT RED	45	43690196	SHROUD, MOTOR, BLACK
16	44689074	OUTER TIRE	30	30519491	REAR SHROUD ASSY, SILVER			

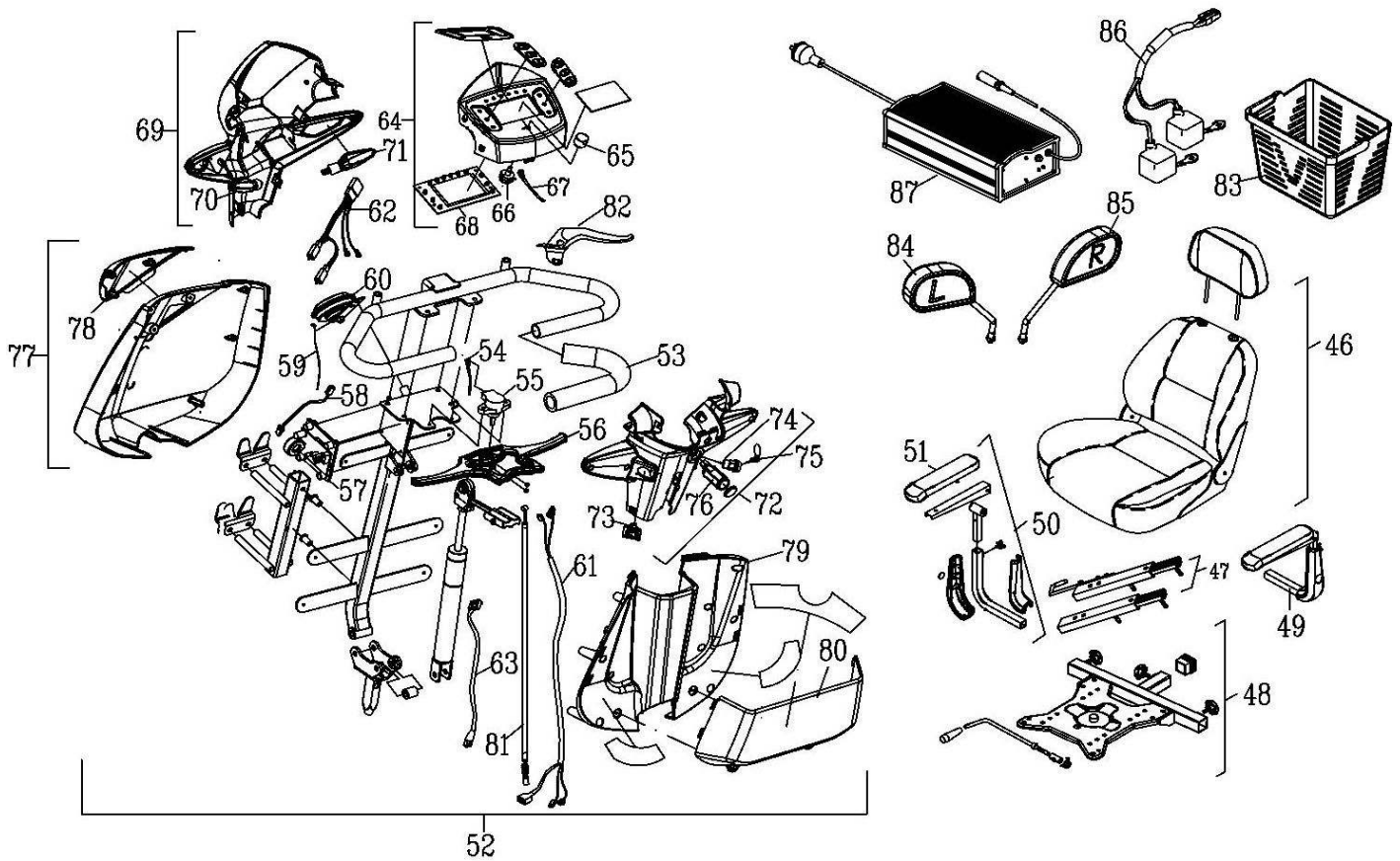

REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION	REF#	PART NO.	DESCRIPTION
46	32228042	CAPTAIN SEAT ONLY, WITH/ RECLINE & MC7/MC4 COLOR	60	61170007	HORN ONLY	74	63700036	KEY SWITCH W/CONNECTING WIRE, S830
47	41700077	SLIDE MECH. ASS'Y., WITH RELEASE LEVER	61	63180033	MAIN CABLE FOR TILLER	75	33700217	KEY SET
48	32100039	SEAT BASE WITH RELEASE LEVER ASSY	62	63453016	HEAD/TURN LIGHT CABLE	76	61138003	CABLE, PLUG.
49	31000325	ARMREST ASS'Y., LH	63	63742003	SENSOR, HALF SPEED	77	30541081	FRONT TILLER SHROUDASSY BRIGHT RED
50	31000326	ARMREST ASS'Y., RH	64	30500279	UPPER DASHBOARD ASSY	77	30519458	FRONT TILLER SHROUDASSY SILVER
51	43701071	ARMREST PAD (DESK ARM), BLACK	65	61180189	KNOB	78	43601374	FRONT WINDSHIELD COVER
52	33500308	TILLER ASS'Y BRIGHT RED	66	61020006	POTENTIOMETER., VR30K, COSMOS RV24	78	43690198	REAR TILLER SHROUD
52	33500309	TILLER ASS'Y SILVER	67	63500018	CONNECTING WIRE, SPEED POT	80	43690200	STORAGE COVER
53	45700102	PU FOAM COATING	68	61108235	LCD PCB BOARD	81	41780024	BRAKE CABLE ASSY
54	63500021	CONNECTING WIRE (POT)	69	30541066	FRONT TILLER SHROUD ASSY, BRIGHT RED,S940	82	42700041	LEVER BRAKE, RH
55	61010047	POTENTIOMETER	69	33500170	FRONT TILLER SHROUD ASSY., SILVER	83	43780088	BASKET, PLASTIC
56	43601255	THROTTLE LEVER SET	70	61200179	TURN LIGHT(L)	84	46781041	LEFT SIDE MIRROR,FO-297,2 TONE
57	61100069	POWER CONVERT BOARD	71	61200180	TURN LIGHT(R)	85	46781042	RIGHT SIDE MIRROR,FO-297,2 TONE
58	63680016	CABLE, POWER CONVERT	72	30500277	REAR DASHBOARD ASSY	86	63270424	BATTERY CONNECTION CABLE, SP64H
59	63500036	HORN WIRE	73	33700256	CHARGER SOCKET SET	87	65000084	CHARGER, HP0180WB(5A)